


# LifeFlow® Quick Guide


1

Attach Y-Connector.

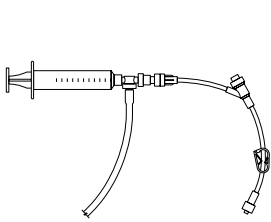


2

Spike IV bag.

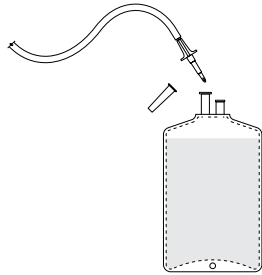


3

Insert syringe with numbers up.

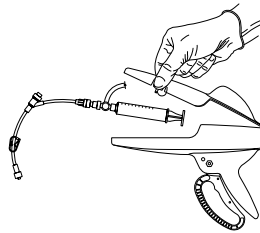


4

Place syringe plunger in blue shuttle, syringe flange into slot.

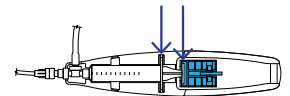


5

Purge all air from bag by cycling trigger.

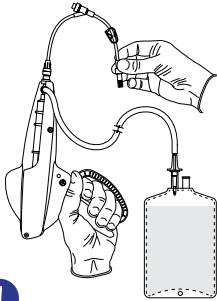


6

Point handle up and fully prime tubing and syringe.

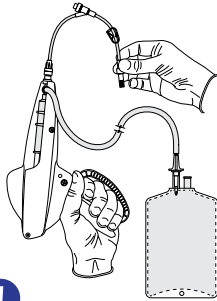


7

Infuse fluids by squeezing the trigger.

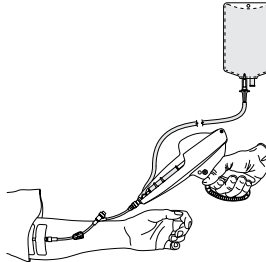

8

Prior to switching bags, disconnect from patient. Then repeat steps 5-7 with new bag.

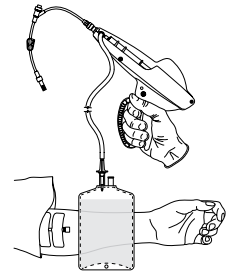

**!** ENSURE COMPLETE REMOVAL OF AIR FROM THE SYSTEM PRIOR TO CONNECTING TO PATIENT.

PLEASE CONSULT THE INSTRUCTIONS FOR USE FOR CAUTIONS, WARNINGS, AND OTHER IMPORTANT INFORMATION.