


Quick Guide


1

Spike bag. Allow AirCheck™ to fill completely. Blue ball should be at the top of AirCheck. If not, squeeze below ball to release.

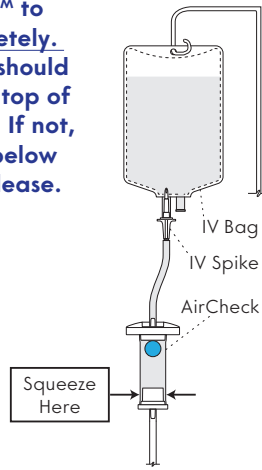


2

Place syringe plunger in blue slot and insert syringe with numbers up.

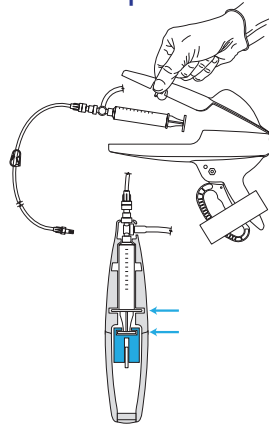


3

Point handle up and fully prime tubing by repeatedly squeezing the trigger.

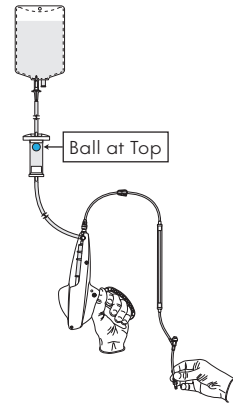

4

Attach to patient. Infuse fluids by repeatedly squeezing trigger. *If using 24G, see note below.

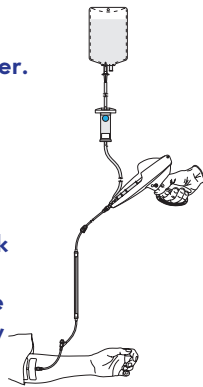


5

Stop infusing before AirCheck is emptied of fluid. To change bags, spike new bag and allow AirCheck to fill completely before infusing.

Resetting AirCheck During Use


IF

The ball is at the bottom of AirCheck, disconnect from the patient.

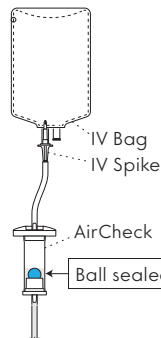

THEN

1. Spike a new bag and wait for AirCheck to fill completely.
2. Squeeze below ball to release.
3. Check for air in the line. Re-prime line if needed, then reconnect.

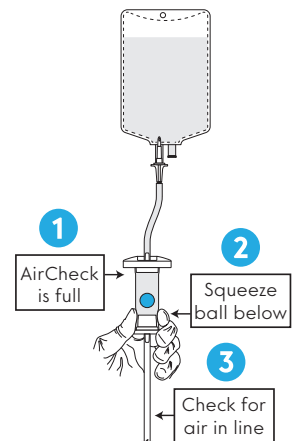

*Note: With 24G and other small catheters, allow a brief pause between trigger squeezes to limit resistance. If resistance increases, allow for a longer pause.

! Important - Do not use with non-power injectable ports, PICCs, and CVCs